



SURGICAL SET CARE GUIDE

✓ Removal of Products from the Set

All products in the surgical set are carefully removed.

✓ Separation of the Set into Stages

The lid, middle and bottom parts of the set are separated from each other.

- Ultrasonic Washing Preparation: The ultrasonic washing device is filled with pure water.
- A lid-sized surgical fluid is added for each set to be washed.
- Ultrasonic Washing: The products are separated into the lid, middle and bottom parts of the set and placed in the ultrasonic washing device.
- The washing process is carried out for 30 minutes.

✓ Ultrasonic Rinsing

After the washing process is completed, the products are rinsed in the ultrasonic device with only pure water for 30 minutes.

✓ Drying

The rinsed products are taken to the oven device and the drying process is carried out.

✓ Reassembly of the Set

The moisture of the lid, middle and bottom parts of the set is removed and the products are placed inside them appropriately.

✓ Packaging

The set is packaged with an autoclave bag.

✓ Sterilization

The packaged set is sterilized in the autoclave device for 1 hour.

✓ Preparation for Use

After the sterilization process is completed, the set is ready for use.